



EASYPOD® HOW TO...

CHANGE THE DOSE

Always check that the dose is correct at the top of the easypod main screen.

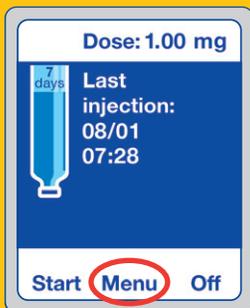


- 1 Power on the easypod device, then press the **MENU** button on the main screen.

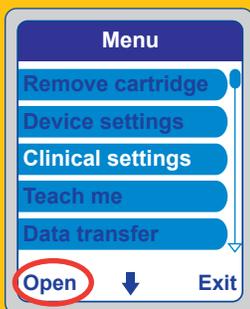

- 2 Use the arrow key to move down and choose **CLINICAL SETTINGS**, then press **OPEN**.



- 3 Press **OK** to the warning message. Enter PIN code **0000** by using the arrow key to select **0** and press **OK**. Repeat until you see **0000**, then press **OK**.

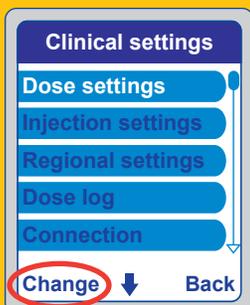

- 4 Select **DOSE SETTINGS**, then press **CHANGE**.

- 5 Choose **INPUT IN MG**, then press **OK**.

- 6 Change the dose using the arrow buttons and press **OK** after each change.

- 7 Confirm frequency, then press **OK**. Confirm dose, then press **OK**.

- 8 Dose adjustment: Press **OK**.

If dose adjustment was OFF, after pressing **OK** you will see a screen asking you to confirm daily dose.

If dose adjustment was ON, after pressing **OK** you will see a dose spread screen with **<10%** or **<25%**. Ensure the correct value is highlighted in white, then press **OK**. Once the screen displays minimum and maximum dose spread, press **OK**.

The dose has now been updated and you are back in the Clinical Settings menu.

If you do not need to make any other changes, press **BACK** again, then press **EXIT** to return to the main screen.



For complete information refer to the **easypod Instructions for Use.**



MEMBER OF
INNOVATIVE MEDICINES CANADA

easypod® is a registered trademark of Merck KGaA. All rights reserved.
© 2020 EMD Serono, a division of EMD Inc., Canada, a business of Merck KGaA, Darmstadt, Germany.

**EMD
SERONO**